- I - SISTEMA DI DRENAGGIO URINA / LIQUIDI DI SCARTO E RELATIVI ACCESSORI - NON STERILE
- GB - URINE/WASTE FLUID DRAINAGE SYSTEM AND RELEVANT ACCESSORIES - NON-STERILE
- F - SYSTEME DE DRAINAGE URINE / LIQUIDES DE DECHET ET ACCESSOIRES  
CORRESPONDANTS - PAS STERILE
- D - DRAINAGESCHLAUCHSYSTEM FÜR URIN / ABFALLFLÜSSIGKEITEN UND ZUBEHÖR-  
UNSTERIL
- E - SISTEMA DE DRENAJE ORINA / LÍQUIDOS DE DESCARTE Y ACCESORIOS  
CORRESPONDIENTES - NO ESTÉRIL
- P - SISTEMA DE DRENAGEM DE URINA / LÍQUIDOS DE ELIMINAÇÃO  
E RESPECTIVOS ACESSÓRIOS - NÃO ESTÉRIL
- N - ANORDNING TIL OPPSAMLING AV URIN / AVFALLSVÆSKE  
OG TILBEHØR - IKKE STERIL
- NL - DRAINSYSTEEM VAN URINE/AFVALVLOEISTOFFEN EN  
BIJBEHORENDE ACCESSOIRES - NIET STERIEL
- FIN - VIRTSAN / HUKKANESTEIDEN VALUTUSJÄRJESTELMÄ  
JA LISÄVARUSTEET - EI STERIILI
- S - DRÄNERINGSSYSTEM FÖR URIN / AVFALLSVÄTSKOR  
MED TILLBEHÖR - INTE STERIL
- DK - ANORDNING TIL OPSAMLING AF URIN / AFFALDSVÆSKE  
OG TILBEHØR - IKKE STERIL
- PL - WOKI DO ZBIÓRKI MOCZU / PŁYNÓW ORGANICZNYCH.  
SYSTEMY DO DRENAŻU I WYPOSAŻENIE DODATKOWE - NIEJAŁOWE

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## APPLICATIONS

Urine/waste fluid collection and approximate measurement of the drained fluids.

## INSTRUCTIONS FOR USE:

- Remove the bag from the packet and thoroughly wash the hands.
- Fix the bag to the right-hand or left-hand side of the bed by means of the handle or use the clamp (if present). The bag must be fixed in a position not higher than the bladder of the patient.
- Put on a pair of disposable gloves (not provided).
- Remove the protective cap from the catheter cone and connect it to the urological catheter (not provided) already inserted in the patient being careful not to contaminate it.
- Take off the gloves and discard them.
- Ensure that the tube is positioned so as to allow regular urine/liquid flow. To do this use the clip if present.
- To read the quantity of uid collected use the approximate graduated scale on the bag as reference.
- When using bag models which have closing devices on the drainage tube (e.g. clamps, tube-clamp plates (3), roller clamps) bladder gymnastics can be carried out.

## HOW TO DRAW A URINE SAMPLE

## Models with drawing point:

- Inform the patient and clamp the drainage tube beyond the drawing point.
- Thoroughly wash the hands.
- Put on a pair of disposable gloves (not provided).
- Disinfect the drawing point and insert the syringe needle (not provided) in the drawing point of the catheter cone.
- Aspirate the required quantity of urine and reopen the tube to restore drainage to the bag.

## Models without drawing point:

- Inform the patient.
- Thoroughly wash the hands.
- Put on a pair of disposable gloves (not provided).
- Draw the sample for analysis using the drainage tap and drain the urine into a clean container (not provided) being careful not to touch the edges of the container.
- When the operation has been completed, close the tap, disinfect it and replace it in its storage pocket (if present).

## HOW TO EMPTY THE BAG

## Models with drainage tap:

- Put on a pair of disposable gloves (not provided).
- Open the drainage tap and wait for the fluid in the bag to completely flow into the collection system adopted (container not provided).
- When the operation has been completed, close the tap, disinfect it and replace it in its storage pocket (if present).
- Put on a pair of disposable gloves (not provided).
- Open the plunger drainage tap holding the fixed part of the tap firmly with the index finger and thumb.
- With the other hand slowly pull and at the same time turn the mobile plunger downward until the end of its travel.
- Wait for the fluid in the bag to completely flow into the collection system adopted (container not provided).
- When the operation has been completed, close the tap firmly holding the fixed part, and with the other hand slowly push and at the same time turn the mobile plunger upward until the end of its travel. Disinfect it.

## Models without drainage tap:

- Put on a pair of disposable gloves (not provided).
- Disconnect the bag and, if necessary, replace it as described in the following point.
- Drain the bag into a container (not provided) or in suitable drain trays rupturing it starting from the pre-cutting position on the device.

## HOW TO REPLACE THE BAG

- Put on a pair of disposable gloves (not provided).
- Extremely gently disconnect the bag from the urological catheter of the patient, repositioning the protective cap on the catheter cone and dispose of the device.
- To apply a new bag on the patient, repeat the operations described previously in the INSTRUCTIONS FOR USE.

## WARNINGS:

- Usage allowed only to qualified physicians. BIOSERVICE declines every responsibility for damages caused to the patient when the device is used in an improper way, with wrong manoeuvres or if handled by non qualified personnel.
- During handling of the device always use a sterile procedure in order to reduce the risks of infection.
- During use never fix the bag in a position higher than the bladder of the patient to prevent backflow into the bladder.
- Do not create traction on the catheter since this is a certain source of internal injury to the urethra.
- Do not use the bag for infusion.
- Do not fill the bag over its maximum capacity.
- The scale on the bag is expressed in millilitres (ml) and is approximate.
- Ensure that the tube is positioned so as to allow regular fluid flow, thus preventing backflow into the bladder.
- Do not place the bag on the floor when it is connected to the catheter, since this is a certain source of infection and malfunctioning of the bag.
- Maximum utilisation time: 48 hours. Depending on the clinical picture of the patient, the medical/nursing staffs responsible for defining the need for more frequent replacement of the device.
- Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user.
- Pyrogen-free
- Expiry: 5 years with undamaged packaging and if stored in normal storage conditions ( keep in a fresh and dry place, avoid the exposure to light and high temperature)
- To safeguard personnel, when required by hospital regulations or by the department, it might be necessary to wear sterile gloves, masks and protective glasses (Warning: these accessories are not supplied with the device).
- The urological catheter is not included in the product.
- Dispose of after use in conformity with the legislation in force.

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